



<b>Display Live Image in Eye Tools</b>	A live image is displayed in the Eye Tools toolbar. If this is not checked, a black box is displayed in the Eye Tool toolbar.
<b>Zoom Magnification</b>	The level of zoom for the magnification tool in the Eye Tools toolbar.
<b>Horizontal/Vertical Button Size</b>	The size of the buttons in the Eye Tools toolbar. This will also change the overall size of the toolbar.
<b>Enable Eye Control On Startup</b>	Eye control is enabled automatically when Quick Glance is started. If not checked, eye control can be enabled from the main Quick Glance screen.
<b>Enable Auto Pilot</b>	Enables Quick Glance to automatically calibrate and operate when a user's eye is detected. This function is designed for quick low accuracy use by multiple users. Currently designed for single eye tracking only.
<b>Update Rate</b>	The number of video frames per second processed. The more frames, the faster Quick Glance will track the eye. More frames may bog down slower computers.
<b>Smoothing Factor</b>	The number of video frames combined. This will help control "jumpiness" of the cursor. The more frames combined will increase lag time of the cursor.
<b>Time between Calibration Targets</b>	The time delay between each calibration target.
<b>Enable Eye Science SDK</b>	Allows direct access from other programs. Not available without the purchase of Eye Science.