


TM5 - Hardware Reference

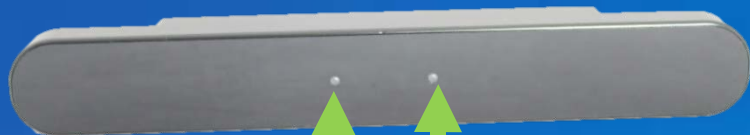

Tracking
Indicators

Front View



Mini USB Jack

End View



Mounting
Holes

Mini USB
Jack

Back View

TM5 - Hardware Quick Start

- 1** Attach your **EyeTech TM5** module to your desired mount using the included Allen wrench and mounting screws.



- 2** Attach your **TM5** and its mount to your monitor, tablet pc or laptop.



- 3** Make certain that the **TM5** is placed just below the monitor and aligned with the middle of the screen.



- 4** Insert the smaller end of the **USB Mini B cable** into the bottom port on the back of the **TM5**.



- 5** Insert the other end of the USB Cable directly into an Available USB port on your computer. **Do NOT plug the cable into a USB hub as this will not provide enough power to operate the TM5.**

