


VT3 - Hardware Reference



Front View



End View

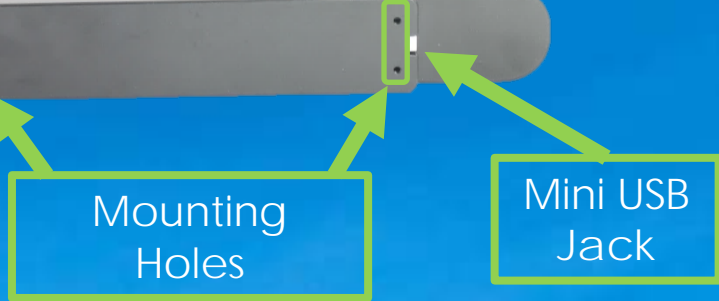

Back View

VT3 - Hardware Quick Start

1 Attach your **EyeTech VT3** module to your desired mount using the included Allen wrench and mounting screws.



2 Attach your **VT3** and its mount to your monitor, tablet pc or laptop.



3 Make certain that the **VT3** is placed just below the monitor and aligned with the middle of the screen.



4 Insert the smaller end of the **USB Mini B cable** into the bottom port on the back of the **VT3**.



5 Insert the other end of the USB Cable directly into an Available USB port on your computer. **Do NOT plug the cable into a USB hub as this will not provide enough power to operate the VT3.**

