


VT3-Mini₁₀ - Hardware Reference

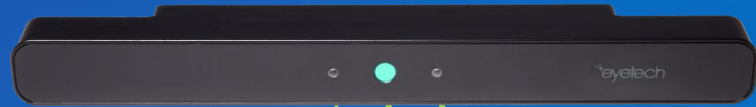

NEW!
Special
Optical Filter
Allows For
Operation
Outdoors and
Near Windows

Tracking
Indicators

Front View



Mini USB Jack

End View



Label

Mounting For
Magnetic or Screw
Attachment

Back View

Contact Us: support@eyetechds.com

1-888-5-EYETECH (1-888-539-3832) 480-704-3158

www.eyetechds.com

VT3-Mini₁₀ - Hardware Quick Start

1 The **VT3-Mini** mounts magnetically to the included metal mounting plate. Remove the paper from both of the adhesive strips on the mounting plate, then apply the mounting plate to the bottom of your laptop, tablet, or monitor's bezel, aligning to the middle of the screen. Make certain that the mounting plate is positioned below the monitor (never on top).



2 Attach your **VT3-Mini** to the mounting plate – the magnets will hold the eye tracker securely in place!



3 Connect the **VT3-Mini** directly to your computer using the included USB cable. If you have a Y cable, use the black end. On some computers you may need to connect both red and black connectors to the computer to get enough power. Never connect through a USB hub as this may not provide enough power.

